


Read the entire label only. Single-Use. Not reuse with natural rubber latex conditions. Store in dry.

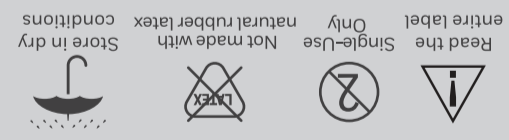

50
Pcs / Box

FLAME SPREAD Class 1 Class 1 Class 1

DELTA P mmHg/cm²

< 5.0 < 6.0 < 6.0

ASTM LEVEL 3
Honeywell

ASTM F2100

PFE @ 0.1 micron

> 95% > 98% > 98%

BFE

> 95% > 98% > 98%

FLUID RESISTANCE mmHg

80 120 160

ASTM LEVEL 1

ASTM LEVEL 2

ASTM LEVEL 3



SURGICAL MASK WITH EARLOOPS
559250P


Box part Number 121820

DT

DT

Additional label will include:
[UDI Barcode]
Lot Number:
Manufacture Date:
Expiration Date:

SURGICAL MASK WITH EARLOOPS

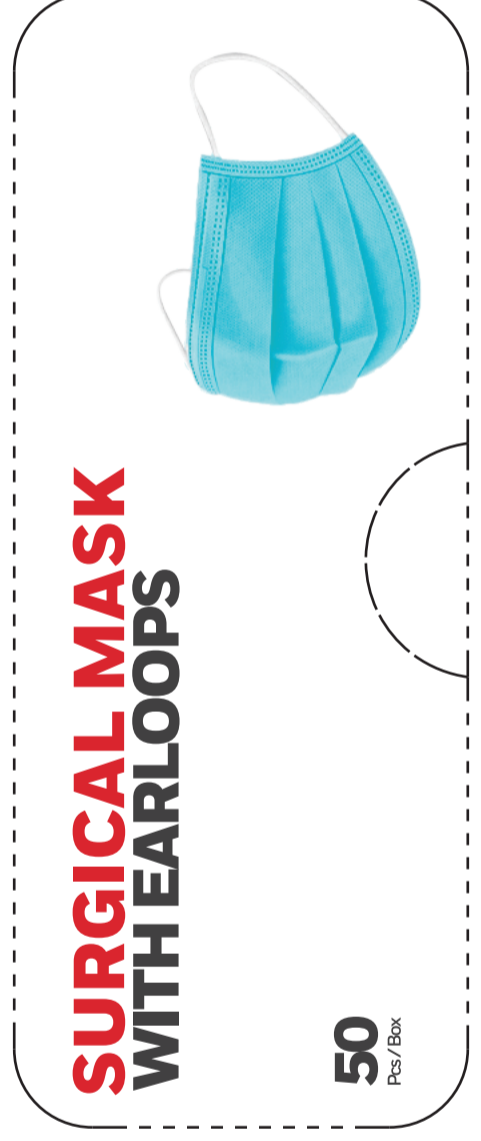

50
Pcs / Box

Honeywell


SURGICAL MASK WITH EARLOOPS
559250P

ASTM LEVEL 3
Honeywell

50
Pcs / Box




DONNING

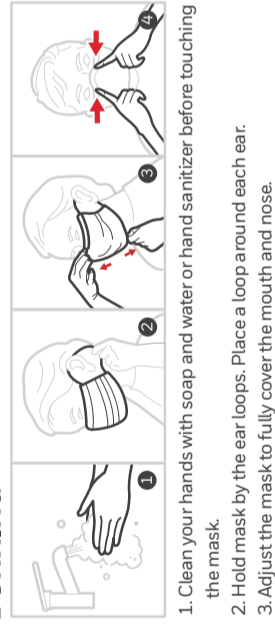

1. Clean your hands with soap and water or hand sanitizer before touching the mask.
2. Hold mask by the ear loops. Place a loop around each ear.
3. Adjust the mask to fully cover the mouth and nose.
4. Form the nose wire around nose and cheek to close any gaps.

DOFFING

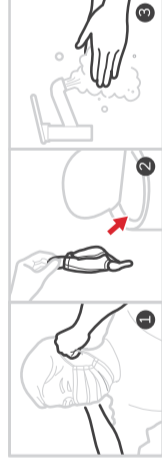

1. Grasp the top edge of the mask by the ear loops and remove mask. Do not touch the front or inside of the mask.
2. Pull mask away and deposit in waste container.
3. Wash your hands or use hand sanitizer.

WARNINGS

Do not use this mask if the mask is damaged or soiled, or if breathing through the mask becomes difficult, you should remove the face mask, discard it safely, and replace with a new one. To safely discard your mask, place it in a plastic bag and put it in the trash. Wash your hands after handling the used mask.

Intended Use:
The Disposable Surgical Face Masks are intended to be worn to protect both the wearer and the person being cared for from the spread of respiratory fluids and particulate material. These face masks are intended for use in infection control practices to reduce the potential exposure to blood and body fluids. This is a single use, disposable device provided non-sterile.



Manufactured for:
Honeywell UIC Healthcare
Surgical Face Mask
Scan for technical information
or visit www.honeywell.com



Made in U.S.A.



Single-Use Only